DE Heizkissen Gebrauchsanweisung.....	2	IT Termoforo Istruzioni per l'uso	31
EN Heat pad Instruction for Use.....	10	TR Isi yastığı Kullanma Talimatı	38
FR Coussin chauffant Mode d'emploi	17	RU Электрическая грелка Инструкция по применению	45
ES Almohadilla eléctrica Instrucciones para el uso	24	PL Poduszka elektryczna Instrukcja obsługi	52

Inhalt

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Contents

1. Items included in the package 11	4.4 Setting the temperature..... 13
1.1 Description 11	4.5 Automatic switch-off 14
2. Important instructions	4.6 Switching off 14
Retain for future use 11	5. Cleaning and maintenance..... 14
3. Intended use..... 13	6. Storage 15
4. Operation..... 13	7. Disposal..... 15
4.1 Safety..... 13	8. What if there are problems? 15
4.2 Initial use 13	9. Technical data 15
4.3 Switching on 13	10. Warranty / Service 15

Explanation of symbols

	Read the instructions		30°C extra gentle washing program
			Do not bleach
	Do not insert pins		Do not tumble dry
			Do not iron
	Do not use folded or rucked		Do not dry-clean
	Not to be used by very young children (0 - 3 years).		
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE		
	Manufacturer		
	CE labelling This product satisfies the requirements of the applicable European and national directives.		


	Certification symbol for products that are exported to the Russian Federation and members of the CIS
	Protection class II device The device is double-protected and thus corresponds to protection class 2
	WARNING: Warning of risks of injury or health hazards
	CAUTION: Safety information about possible damage to appliance/accessories.
	NOTE: Important information.

1. Items included in the package

- 1 Heating pad
- 1 Controller
- 1 Instructions for use

1.1 Description

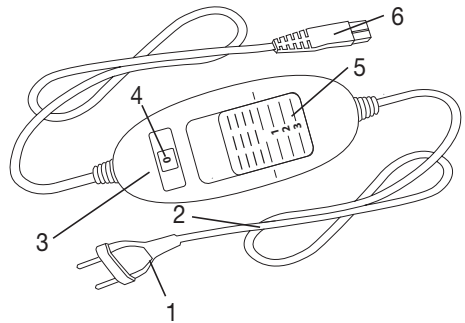- 1. Mains plug
- 2. Mains cable
- 3. Controller
- 4. Illuminated display for temperature settings
- 5. Sliding switch for ON/OFF and temperature settings
- 6. Plug-in coupling

2. Important instructions

Retain for future use

WARNING

- Non-observance of the following notes may result in personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information is not only intended to protect your health and the health of others, it should also protect the product. For this reason, pay attention to these safety notes and include these instructions when handing over the product to others.
- This heating pad must not be used by people who are not sensitive to heat or by vulnerable people who may not be able to react to overheating (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any pain-relieving medication or alcohol).

- This heating pad may not be used by very young children (0-3 years old) as they are unable to respond to overheating.
- This heating pad may not be used by young children (3-8 years old) unless the controller has been set by a parent or responsible adult and the child has been given adequate instructions on how to use the heating pad safely.
- This heating pad may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised and have been instructed on how to use the heating pad safely, and are fully aware of the consequent risks of use.
- Children must not play with the heating pad.
- Cleaning and user maintenance must not be performed by children unless supervised.
- This heating pad is not designed for use in hospitals.
- Do not insert needles.
- Do not use when folded or bunched up.
- Do not use if wet.
- This heating pad may only be used in conjunction with the controller specified on the label.
- The electrical and magnetic fields emitted by this heating pad may interfere with the function of a pacemaker. However, they are still well within the limits: electrical field strength: max. 5000 V/m, magnetic field strength: max. 80 A/m, magnetic flux density: max. 0.1 millitesla. Please consult your doctor and the manufacturer of your pacemaker before using this heating pad.
- Do not pull, twist or make sharp bends in the cables.
- Please check this heating pad frequently for signs of wear and tear or damage.

If any such signs are evident, if the heating pad has been used incorrectly or if it no longer heats up, it must be checked by the manufacturer before being switched on again.

- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- While the heating pad is switched on, the following conditions must be observed:
 - Objects must not be placed on the pad (e.g. suitcase or washing basket).
 - Heat sources, such as hot water bottles, heating pads or similar, must not be placed on the pad.

- The electronic components in the controller warm up when the heating pad is in use. For this reason, the controller must never be covered or placed on the heating pad when it is in use.
- Always observe the notes on operation (section 4), cleaning and maintenance (section 5) and storage (section 6).
- If you should have any questions about using our devices, please contact Customer Services.

3. Intended use

CAUTION

This heating pad is only designed to warm up the human body.

4. Operation

4.1 Safety

CAUTION

The heating pad is fitted with a SAFETY SYSTEM. This sensor technology provides protection against overheating across the entire surface of the heating pad with an automatic switch-off in the event of a fault. If the SAFETY SYSTEM has switched off the heating pad, the temperature settings are no longer illuminated when switched on. Please note that for safety reasons, the heating pad can no longer be operated after a fault has occurred and must be sent to the specified service address.

Do not connect the defective heat pad with another controller of the same type. This would trigger a permanent switch-off via the controller safety system.

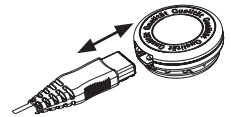
4.2 Initial use

CAUTION

Ensure that the heating pad will not bunch up or become folded during use.

- First, connect the controller to the heating pad by inserting the plug-in coupling.
- Then plug the mains plug into the socket.

4.3 Switching on

To switch the heating pad on, set the sliding switch for ON/OFF and temperature settings to setting 1, 2 or 3. The temperature settings (4) are illuminated when switched on.

4.4 Setting the temperature

Level 0: OFF

Level 1: minimum heat

Level 2: medium heat

Level 3: maximum heat

NOTE:

The fastest way to warm up the heating pad is to initially set the highest temperature setting.

NOTE:

These heating pads have a fast heating function, which allows the pad to warm up faster in the first 10 minutes.

WARNING

If the heating pad is being used over several hours, we recommend that you set the lowest temperature setting on the controller to avoid overheating the heated body part, which may lead to burns to the skin.

4.5 Automatic switch-off

This heating pad is equipped with an automatic switch-off function. This turns off the heat supply approx. 90 minutes after the initial use of the heating pad. The temperature setting display then starts to flash. So that the heating pad can be switched back on, the sliding switch for ON/OFF and temperature settings (5) must first be set to setting "0" (OFF). After about 5 seconds it is possible to switch it on again.

4.6 Switching off

To switch the heating pad off, set the sliding switch for ON/OFF and temperature settings to setting "0" (OFF). The temperature settings are then no longer illuminated.

NOTE:

If the heating pad is not in use, switch the side sliding switch for ON/OFF to setting "0" (OFF) and unplug the mains plug from the socket. Then disconnect the controller from the heating pad by unplugging the plug-in coupling.

5. Cleaning and maintenance

WARNING

Before cleaning, always remove the mains plug from the socket first. Then disconnect the controller from the heating pad by unplugging the plug-in coupling. Otherwise there is a risk of electric shock.

CAUTION

The controller must never come into contact with water or other liquids, as this may cause damage.

To clean the controller, use a dry, lint-free cloth. Do not use any chemical or abrasive cleaning agents.

Small marks on the heating pad can be removed with a damp cloth and if necessary, with a little liquid detergent for delicate laundry.

CAUTION

Please note that the heating pad may not be chemically cleaned, wrung out, tumble dried, put through a mangle or ironed.

This heating pad is machine-washable. Set the washing machine to a particularly gentle wash cycle at 30°C (wool cycle). Use a delicate laundry detergent and measure it out according to the manufacturer's instructions.

CAUTION

Please note that frequent washing of the heating pad has a negative effect on the product. The heating pad should therefore be washed in a washing machine a maximum of 10 times during its life.

Immediately after washing, reshape the heating pad to its original dimensions while it is still damp and spread it out flat on a clothes horse to dry.

CAUTION

- Do not use pegs or similar items to attach the heating pad to the clothes horse. Otherwise the heating pad may be damaged.
- Do not reconnect the switch to the heating pad until the plug-in connection and the heating pad are completely dry. Otherwise the heating pad may be damaged.

WARNING

Never switch the heating pad on to dry it! Otherwise there is a risk of electric shock.

6. Storage

If you do not plan to use the heating pad for a long period of time, we recommend that you store it in the original packaging. For this purpose, disconnect the controller from the heating pad by unplugging the plug-in coupling.

CAUTION

Please allow the heating pad to cool down before storing. Otherwise the heating pad may be damaged. To avoid sharp folds in the heating pad, do not place any objects on top of it while it is being stored.

7. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

8. What if there are problems?

Problem	Cause	Solution
The temperature settings are not illuminated while - the controller is properly connected to the heating pad - the mains plug is connected to a working socket - the sliding switch is set to 1, 2 or 3	The safety system has switched the heating pad off permanently.	Send the heating pad and the controller for servicing.

9. Technical data

See the rating label on the heating pad.

10. Warranty / Service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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