<b>DE</b> Blutdruckmessgerät Gebrauchsanweisung .....	2
<b>EN</b> Blood pressure monitor Instructions for use .....	22
<b>FR</b> Tensiomètre Mode d'emploi .....	40
<b>ES</b> Tensiómetro Instrucciones de uso .....	60
<b>IT</b> Misuratore di pressione Istruzioni per l'uso .....	80
<b>TR</b> Tansiyon ölçme cihazı Kullanım kılavuzu .....	100
<b>RU</b> Прибор для измерения артериального давления на запястье Инструкция по применению .....	117
<b>PL</b> Ciśnieniomierz Instrukcja obsługi .....	137

## Inhalt

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## Contents

1. Getting to know your device .....	22	7. Saving, displaying and deleting measured values .	33
2. Important notes .....	23	8. Error messages/troubleshooting .....	36
3. Unit description .....	27	9. Cleaning and storing the device and cuff .....	36
4. Preparing for the measurement .....	28	10. Technical specifications .....	36
5. Measuring blood pressure .....	29	11. Warranty/service .....	38
6. Evaluating results .....	32		

## Included in delivery

- Blood pressure monitor with cuff
- 2 x 1.5 V LR03 AAA batteries
- Storage box
- Instructions for use

## Dear customer,

thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty and air. Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards, your Beurer team

## 1. Getting to know your device

Check that the device packaging has not been tampered with and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

The wrist blood pressure monitor is used to carry out non-invasive measurement and monitoring of arterial blood pressure values in human adults. This allows you to quickly and easily measure your blood pressure, save the measurements and display the development of the measurements. You are also warned of possible existing cardiac arrhythmia.

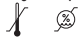
The recorded values are classified and evaluated graphically.

## 2. Important notes

### Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and the accessories:

	Attention
	Note Note on important information
	Observe the instructions for use
	Application part, type BF
	Direct current
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Dispose of packaging in an environmentally friendly manner

	Manufacturer
	Permissible storage and transport temperature and humidity
	Permissible operating temperature and humidity
	Protect from moisture
	Serial number
	<b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives.

### Notes on use

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.
- Before the initial blood pressure measurement, make sure always to rest for about 5 minutes.

- Furthermore, if you want to take several measurements in succession, make sure always to wait for at least 1 minute between the individual measurements.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are no substitute for a medical examination.
- Discuss the measurements with your doctor and never base any medical decisions on them (e.g. medicines and their administration).
- Using the blood pressure monitor outside your home environment or whilst on the move (e.g. whilst travelling in a car, ambulance or helicopter, or whilst undertaking physical activity such as playing sport) can influence the measurement accuracy and cause incorrect measurements.
- Do not use the blood pressure monitor on newborns or patients with pre-eclampsia. We recommend consulting a doctor before using the blood pressure monitor during pregnancy.
- In the case of restricted circulation on the arm as a result of chronic or acute vascular diseases (including vascular constriction), the accuracy of the wrist measurement is limited. In this case you should avoid using an upper arm blood pressure monitor.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device. Supervise children around the device to ensure they do not play with it.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the unit on people who have the specified wrist measurement for the device.
- Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions, remove the cuff from the arm.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Ensure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment,

e.g. intravascular access or therapy, or an arterio-venous (AV) shunt.

- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- Place the cuff on your wrist only. Do not place the cuff on other parts of the body.
- The blood pressure monitor can only be operated with batteries.
- To conserve the batteries, the blood pressure monitor switches off automatically if you do not press any buttons for 30 seconds.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.
- Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty, the blood pressure monitor loses the date and time.

### **Instructions for storage and maintenance**

- The blood pressure monitor is made from precision and electronic components. The accuracy of the measurements and service life of the device depend on its careful handling:

- Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
- Do not drop the device.
- Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.
- We recommend that the batteries be removed if the device will not be used for a prolonged period of time.

### **Notes on handling batteries**

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
-  **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.

- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

### **Instructions for repairs and disposal**

- Batteries do not belong in household waste. Please dispose of empty batteries at the collection points intended for this purpose.
- Do not open the device. Failure to comply will result in voiding of the warranty.
- Do not repair or adjust the device yourself. Proper operation can no longer be guaranteed in this case.
- Repairs must only be carried out by Customer Services or authorised suppliers. Before making a claim, please check the batteries first and replace them if necessary.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.


### **Notes on electromagnetic compatibility**

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

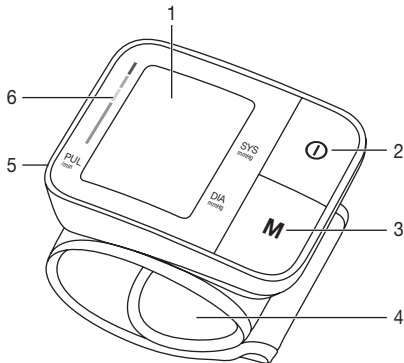
### 3. Unit description

1. Display
2. **START/STOP** button 
3. Memory button **M**
4. Wrist cuff
5. Battery compartment lid
6. Risk indicator

### Information on the display:

1. Risk indicator
2. Symbol for *Bluetooth*<sup>®</sup> transfer 
3. Time and date
4. Systolic pressure
5. Diastolic pressure
6. Calculated pulse value
7. Cardiac arrhythmia symbol   
Pulse symbol 
8. Battery status indicator 
9. Release air (arrow)
10. User memory 
11. Memory space number/memory display  
average value (A), morning (AM), evening (PM)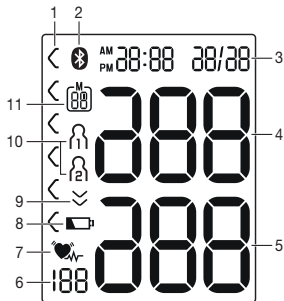

## System requirements for the app “beurer HealthManager”:

- iOS ≥ 10.0, Android™ ≥ 5.0
- Bluetooth® ≥ 4.0


List of compatible devices:

## 4. Preparing for the measurement

### Inserting the batteries

- Remove the battery compartment lid on the left side of the device.
- Insert two 1.5 V micro (alkaline type LR03) batteries. Make sure that the batteries are inserted the correct way round. Do not use rechargeable batteries.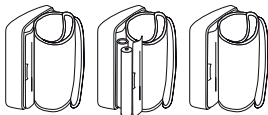

- Close the battery compartment lid again carefully. If the battery change symbol  is flashing and **Lo** appears, no further measurements are possible and you must replace all batteries.

**24h** flashes on the display. Now set the date and time as described below.

- The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
- The codes below are printed on batteries containing harmful substances:  
Pb = Battery contains lead,  
Cd = Battery contains cadmium,  
Hg = Battery contains mercury.

### Set the hour format, date, time and Bluetooth® settings

This menu allows you to set the following functions, one after another.

It is essential to set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

-  If you press and hold the memory button **M**, you can set the values more quickly.

**Press and hold the START/STOP  button for 5 seconds if you have not reinserted the batteries.**

The hour format now flashes on the display.

- Select the desired hour format using the memory button **M** and confirm with the **START/STOP** button **ⓘ**.

The year flashes on the display.

- Select the desired year using the memory button **M** and confirm with the **START/STOP** button **ⓘ**.

The month flashes on the display.

- Select the desired month using the memory button **M** and confirm with the **START/STOP** button **ⓘ**.

The day flashes on the display.

- Select the desired day using the memory button **M** and confirm with the **START/STOP** button **ⓘ**.

- i** If the hour format is set as **12h**, the day/month display sequence is reversed.

The hours flash on the display.

- Select the desired hour using the memory button **M** and confirm with the **START/STOP** button **ⓘ**.

The minutes flash on the display.

- Select the desired minute using the memory button **M** and confirm with the **START/STOP** button **ⓘ**.

The *Bluetooth*® symbol flashes on the display.


- Use the memory button **M** to select whether automatic *Bluetooth*® data transfer is activated (*Bluetooth*® symbol flashes) or deactivated (*Bluetooth*® symbol is not shown) and confirm with the **START/STOP** button **ⓘ**.

- i** *Bluetooth*® transfers will reduce the battery life.

## 5. Measuring blood pressure

Ensure the device is at room temperature before measuring. The measurement can be performed on the left or right wrist.

## Attaching the cuff

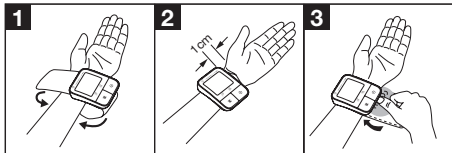- Expose your wrist. Ensure that the circulation of the arm is not hindered by tight clothing or similar.
- Place the cuff on the inside of your wrist.
- Close the cuff with the hook-and-loop fastener such that the upper edge of the device is positioned approx. 1 cm underneath the heel of the hand.
- The cuff must be fitted tightly around the wrist, but must not constrict.

Blood pressure may vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm.

## Adopt the correct posture

- Before the initial blood pressure measurement, make sure always to rest for about 5 minutes. Otherwise deviations can occur.
- Furthermore, if you want to take several measurements in succession, make sure always to wait for at least 1 minute between the individual measurements.
- You can take the measurement while sitting or lying. To take your blood pressure, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground. Make sure to rest your arm and move it. Always make sure that the cuff is at heart level. Otherwise significant deviations can occur. Relax your arm and the palm of your hand.
- To avoid falsifying the measurement, it is important to remain still during the measurement and not to speak.

## Performing the blood pressure measurement

As described above, attach the cuff and adopt the posture in which you want to perform the measurement.

- Press the **START/STOP** button **I** to start the blood pressure monitor. All displays will illuminate briefly.


The blood pressure monitor will begin the measurement automatically after 5 seconds.

The measurement is taken during the inflation phase.


- You can cancel the measurement at any time by pressing the **START/STOP** button **I**.

As soon as a pulse is found, the pulse symbol  is displayed.

- The systolic pressure, diastolic pressure and pulse rate measurements are displayed.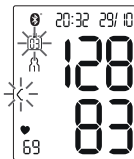

- **EE\_** appears if the measurement could not be performed properly. Observe the chapter on error messages/troubleshooting in these instructions for use and repeat the measurement.
- Now select the desired user memory by pressing the memory button **M**. If you do not select a user memory,

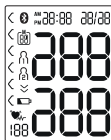the measurement is stored in the most recently used user memory. The relevant symbol **M1** or **M2** appears in the display.

- Press the **START/STOP** button **I** to switch off the blood pressure monitor. The measurement is then stored in the selected user memory.

If **Bluetooth® data transfer** is activated, data is transferred after pressing the **START/STOP** button **I**.

The **Bluetooth®** symbol flashes on the display. The blood pressure monitor now attempts to connect to the app for approx. 30 seconds.

The **Bluetooth®** symbol stops flashing as soon as a connection is established. All measurement data is automatically transferred to the app. Once the data has been successfully transferred, the device switches off.

- If a connection to the app cannot be established after 30 seconds, the **Bluetooth®** symbol goes out and the blood pressure monitor switches off automatically after 3 minutes.
- If you forget to turn off the device, it will switch off automatically after a maximum of 3 minutes. In this case too, the value is stored in the selected or most recently used user memory.
- Wait for at least 1 minute before taking another measurement.

## 6. Evaluating results

### Cardiac arrhythmia:

This device can identify potential disruptions of the heart rhythm when measuring and if necessary, indicates this after the measurement with the symbol .

This can be an indicator for arrhythmia. Arrhythmia is an illness in which the heart rhythm is abnormal because of flaws in the bioelectrical system that regulates the heart-beat. The symptoms (skipped or premature heart beats, pulse being slow or too fast) can be caused by factors such as heart disease, age, physical make-up, excess stimulants, stress or lack of sleep. Arrhythmia can only be determined through an examination by your doctor.

If the symbol  is shown on the display after the measurement has been taken, repeat the measurement. Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol  appears frequently, please consult your doctor. Self-diagnosis and treatment based on the measurements can be dangerous. Always follow your GP's instructions.

### Risk indicator:

The measurements can be classified and evaluated in accordance with the following table.

However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The bar chart on the display and the scale on the unit show which category the recorded blood pressure values fall into. If the values of systole and diastole fall into two different categories (e.g. systole in the 'High normal' category and diastole in the 'Normal' category), the graphical classification on the device always shows the higher category; for the example given this would be 'High normal'.

Blood pressure value category	Systole (in mmHg)	Diastole (in mmHg)	Action
Setting 3: severe hypertension	≥ 180	≥ 110	seek medical attention
Setting 2: moderate hypertension	160–179	100–109	seek medical attention
Setting 1: mild hypertension	140–159	90–99	regular monitoring by doctor
High normal	130–139	85–89	regular monitoring by doctor

Blood pressure value category	Systole (in mmHg)	Diastole (in mmHg)	Action
Normal	120–129	80–84	self-monitoring
Optimal	<120	<80	self-monitoring

Source: WHO, 1999 (World Health Organization)

## 7. Saving, displaying and deleting measured values

### User memory

The results of every successful measurement are stored together with the date and time. The oldest measurement is overwritten in the event of more than 60 measurements.

- Once the device is switched off, select the desired user memory (M1 M2) using the memory button **M** and confirm with the **START/STOP** button **ⓘ**.
- The average value of all measurements appears on the display.

### User memory

- If *Bluetooth*<sup>®</sup> is activated (the  symbol flashes on the display), the blood pressure monitor attempts to connect to the app. If you press the memory button **M** in the meantime, the transfer is cancelled and the average values are displayed. The  symbol is no longer shown. The buttons are deactivated and the  symbol ceases to flash as soon as a connection is established and the data is transferred.

 flashes on the display.

The average value of all saved measured values in this user memory is displayed.

- Press the memory button **M**.

 flashes on the display.

The average value of the morning measurements for the last 7 days is displayed (morning: 5.00 a.m. – 9.00 a.m.).

- Press the memory button **M**.

 flashes on the display.

The average value of the evening measurements for the last 7 days is displayed (evening: 6.00 p.m. – 8.00 p.m.).


### Average values

- When the memory button **M** is pressed again, the last individual measurement is displayed (in this example, measurement 03).
  - When the memory button **M** is pressed again, you can view your individual measured values.
  - To switch the device off again, press the **START/STOP** button ①.
- ① You can exit the menu at any time by pressing the **START/STOP** button ①.

- To clear the memory of the relevant user memory, you must first select a user memory.
- Start the retrieval of the average measured values. **R** flashes on the display and the average value of all saved measured values in this user memory is displayed.
- Press and hold the memory button **M** for 5 seconds depending on the user memory you are in.

All the values in the current user memory are deleted.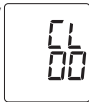

- To clear individual measurements from the relevant user memory, you must first select a user memory.
- Start individual measurement access.
- Press and hold the memory button **M** for 5 seconds (depending on the user memory you are in).
- The selected value is deleted. The device briefly displays **CL 00**.
- If you would like to delete other values, repeat the process described above.

You can switch off the device at any time by pressing the **START/STOP** button ①.

### Transfer via *Bluetooth*®

It is also possible to transfer the measured values saved on the device to your smartphone via *Bluetooth*®.

You will need the “beurer HealthManager” app for this. This is available in the Apple App Store and from Google Play.

Proceed as follows to transfer values:

If *Bluetooth*® is activated in the settings menu, the data is transferred automatically after the measurement. The **BT** symbol appears in the top left of the display (see chapter 4 “Preparing the measurement”).

### Step 1: BC 57

Activate *Bluetooth*® on your device (see chapter 4 “Preparing the measurement, *Bluetooth*®”).

### Step 2: “beurer HealthManager” app

In the “beurer HealthManager” app, add the BC 57 under “Settings/My devices”.

### Step 3: BC 57

take a measurement.

### Step 4: BC 57

data transfer immediately following measurements.

- If **Bluetooth® data transfer** has been activated, data is transferred after having confirmed the user memory and pressing the **START/STOP** button ①.

### Step 4: BC 57

Data transfer at a later point:

- Go to memory mode (chapter 7). Select the desired user memory. The *Bluetooth*® transfer starts automatically.

- ① The “beurer HealthManager” app must be active to allow data transfer.

If your smartphone has a protective cover, remove this to ensure that there is no interference during the transfer. Begin the data transfer in the “beurer HealthManager” app.

We would explicitly draw attention to the fact that the software to hand is not a medical product in accordance with EU Directive 93/42/EEC.

The values displayed are purely for visualisation purposes and must not be used as a basis for therapeutic treatments. The software is not part of a diagnostic-medical system.

## 8. Error messages/troubleshooting

In case of faults, the **EE**\_ message appears in the display. Error messages may appear if:

- systolic or diastolic pressure could not be measured (**EE 1** appears on the display)
- systolic or diastolic pressure was outside the measurement range (**H 1** or **L 0** appears on the display)
- the cuff is fastened too tightly or loosely (**EE 2** appears on the display)
- the pump pressure is higher than 300 mmHg (**EE B** appears on the display)
- if the pressure during pumping up remains below 15 mmHg for longer than 3 minutes (**EE 3** appears on the display)
- there is a system or device error (**EE 0**, **EE 4**, **EE 5** or **EE 6** appears on the display)
- the data could not be sent via *Bluetooth*<sup>®</sup> (**EE 7**).

In such cases, repeat the measurement. Ensure that you do not move or speak during the measurement. If necessary, reinsert or replace the batteries.


## 9. Cleaning and storing the device and cuff

- Clean the device and cuff carefully using a slightly damp cloth only.
- Do not use any cleaning agents or solvents.

- Under no circumstances hold the device and cuff under water, as this can cause liquid to enter and damage the device and cuff.
- If you store the device and cuff, do not place heavy objects on the device and cuff. Remove the batteries.

## 10. Technical specifications

Model no.	BC 57
Measurement method	Oscillometric, non-invasive blood pressure measurement on the wrist
Measurement range	Cuff pressure 0-300 mmHg, Systolic 60-260 mmHg, Diastolic 40-199 mmHg, Pulse 40-180 beats/minute
Display accuracy	Systolic $\pm 3$ mmHg, Diastolic $\pm 3$ mmHg, Pulse $\pm 5$ % of the value shown
Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: Systolic 8 mmHg / Diastolic 8 mmHg
Memory	2 x 60 memory spaces
Dimensions	L 78 mm x W 60 mm x H 23 mm
Weight	Approximately 96 g (without batteries, with cuff)
Cuff size	140 to 195 mm

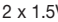Permissible operating conditions	+10 °C to +40 °C, ≤85 % relative humidity (non-condensing)
Permissible storage conditions	-20 °C to +50 °C, ≤85 % relative humidity, 800-1050 hPa ambient pressure
Power supply	2 x 1.5V  AAA batteries
Battery life	For approx. 200 measurements, depending on levels of blood pressure and pump pressure
Data transfer	2402 MHz – 2480 MHz frequency band Transmission power max. 2.8 dBm The blood pressure monitor uses <i>Bluetooth</i> <sup>®</sup> low energy technology Compatible with <i>Bluetooth</i> <sup>®</sup> 4.0 smartphones/tablets  List of supported smartphones/tablets

The serial number is located on the device or in the battery compartment.

Technical information is subject to change without notification to allow for updates.

- This device is in line with European Standard EN 60601-1-2 (In accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3 and IEC 61000-4-8)

and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit.

- This device corresponds to the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktgesetz) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2-30: Particular requirements for the basic safety and essential performance of automated non-invasive sphygmomanometers).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life.
- If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.
- We hereby confirm that this product complies with the European RED Directive 2014/53/EU. The CE Declaration of Conformity for this product can be found under: [www.beurer.com/web/we-landingpages/de/cedeclarationofconformity.php](http://www.beurer.com/web/we-landingpages/de/cedeclarationofconformity.php).

## 11. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.**

**The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 5 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry

out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of

- the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
  - products purchased as seconds or as used goods;
  - consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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- The *Bluetooth*<sup>®</sup> word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer GmbH is under license. Other trademarks and trade names are those of their respective owners.
- Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.
- Google Play and the Google Play logo are trademarks of Google LLC.
- Android is a trademark of Google LLC.