DE Blutdruckmessgerät Gebrauchsanweisung	2
EN Blood pressure monitor Instructions for use	24
FR Tensiomètre Mode d'emploi	44
ES Tensiómetro Instrucciones de uso	65
IT Misuratore di pressione Istruzioni per l'uso	86
TR Tansiyon ölçme cihazı Kullanım kılavuzu	107
RU Прибор для измерения кровяного давления Инструкция по применению	126
PL Ciśnieniomierz Instrukcja obsługi	148
Notes on electromagnetic compatibility	169

DEUTSCH

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Dear customer,

thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty and air.

With kind regards,
your Beurer team

Table of contents

1. Included in delivery	25	7. Usage	32
2. Signs and symbols	25	8. Cleaning and maintenance.....	39
3. Intended Use	26	9. What if there are problems?	39
4. Warnings and safety notes	26	10. Disposal.....	40
5. Device description.....	29	11. Technical specifications	40
6. Initial use	30	12. Warranty / Service	42

1. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 x Wrist blood pressure monitor with cuff
- 1 x Instructions for use
- 1 x Quick Start Guide
- 1 x Storage box
- 2 x 1.5V AAA batteries LR03

2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	WARNING Warning instruction indicating a risk of injury or damage to health
	CAUTION Safety note indicating possible damage to the device/accessory

	Note Note on important information
	Observe the instructions for use
	Application part, type BF
	Direct current
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Do not dispose of batteries containing hazardous substances with household waste
  PAP	Dispose of packaging in an environmentally friendly manner
	Manufacturer
	European Authorised Representative for manufacturers of medical devices
	Permissible storage and transport temperature and humidity

	Permissible operating temperature and humidity
IP22	Protected against solid foreign objects 12.5 mm in diameter and larger, and against drops of water when the housing is angled up to 15°
SN	Serial number
CE 0123	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products

3. Intended Use

The BC 54 wrist blood pressure monitor is intended for home use and serves as a non-invasive way of measuring and monitoring arterial blood pressure values and pulse rates in adults with a wrist circumference of 13.5 – 21.5 cm. As a result, you can quickly and easily measure your blood pressure and pulse and view various average values from previous measurements. The recorded measurements are classified and evaluated graphically. A symbol in the display also warns of possible existing cardiac arrhythmia.

4. Warnings and safety notes

Notes on use

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.
- Before the initial blood pressure measurement, make sure always to rest for about 5 minutes.
- Furthermore, if you want to take several measurements in succession, make sure always to wait for at least 1 minute between the individual measurements.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are no substitute for a medical examination.
- Discuss the measurements with your doctor and never base any medical decisions on them (e.g. medicines and their administration).
- Using the blood pressure monitor outside your home environment or whilst on the move (e.g. whilst travelling in a car, ambulance or helicopter, or whilst undertaking physical activity such as playing sport) can influence the measurement accuracy and cause incorrect measurements.

- Do not use the blood pressure monitor on newborns or patients with pre-eclampsia. We recommend consulting a doctor before using the blood pressure monitor during pregnancy.
- In the case of restricted circulation on the arm as a result of chronic or acute vascular diseases (including vascular constriction), the accuracy of the wrist measurement is limited. In this case you should avoid using an upper arm blood pressure monitor.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device. Supervise children around the device to ensure they do not play with it.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the unit on people who have the specified wrist measurement for the device.
- Please note that when inflating, the functions of the limb in question may be impaired.
- Inflating the cuff can temporarily lead to malfunctions in devices used simultaneously and on the same limb.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions, remove the cuff from the arm.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Ensure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- Place the cuff on your wrist only. Do not place the cuff on other parts of the body.
- The blood pressure monitor can only be operated with batteries.
- Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty, the blood pressure monitor loses the date and time.

- To conserve the batteries, the blood pressure monitor switches off automatically if you do not press any buttons for 30 seconds.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.
- The user must check that the equipment functions safely and see that it is in proper working condition before being used. The manufacturer does not require such preventive inspections by other persons. Manufacturer will provide circuit diagrams, component part lists, descriptions, calibration instructions to assist to service personnel.
- The maximum temperature that the applied part can achieve is 42.8 °C at an environmental temperature of 40 °C.

Notes on storage

- The blood pressure monitor is made from precision and electronic components. The accuracy of the measurements and service life of the device depend on its careful handling:
 - Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
 - Do not drop the device.

- Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.

- We recommend that the batteries be removed if the device will not be used for a prolonged period of time.

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.

-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.

- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.

-  **Risk of explosion!** Never throw batteries into a fire.

- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.


Notes on electromagnetic compatibility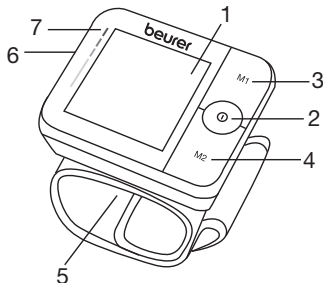

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.


5. Device description

1. Display
2. **START/STOP** button 
3. Memory button **M1**
4. Memory button **M2**
5. Wrist cuff
6. Battery compartment lid
7. Risk indicator

Information on the display:

1. Risk indicator
2. Symbol for *Bluetooth*® transfer
3. Time and date
4. Systolic pressure
5. Diastolic pressure
6. Calculated pulse value
7. Cardiac arrhythmia symbol 
Pulse symbol 
8. Battery status indicator 
9. Release air 
10. User memory  / 
11. Memory space number/memory display
average value (A), morning (AM), evening (PM)

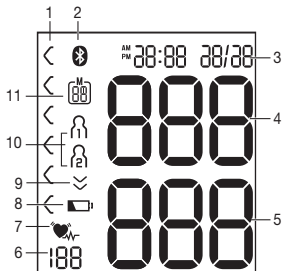6. Initial use

Inserting the batteries

- Remove the battery compartment lid on the left side of the device.
- Insert two 1.5 V AAA micro (alkaline type LR03) batteries. Make sure that the batteries are inserted the correct way round. Do not use rechargeable batteries.
- Close the battery compartment lid again carefully.

Now set the date and time as described below.

If the battery change symbol  is flashing and *ErE* appears, no further measurements are possible and you must replace all batteries. Once the batteries have been removed from the device, the date and time must be set again. Any saved measured values are retained.

Set the hour format, date, time and *Bluetooth*® settings

It is essential that you set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

i There are two different ways to access the menu from which you can adjust the settings:

- Before initial use and after each time you replace the battery:

When inserting batteries into the device, you will be taken to the relevant menu automatically.

- If the batteries have already been inserted:
Press and hold the **START/STOP** button **⓪** on the device when **switched off** for approx. 5 seconds.

In this menu you can adjust the following settings in succession:

It is essential to set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

Hour format

The hour format now flashes on the display.

- Select the desired hour format using the memory button **M1** or **M2** and confirm with the **START/STOP** button **⓪**.

Date

The year flashes on the display.

- Select the desired year using the memory button **M1** or **M2** and confirm with the **START/STOP** button **⓪**.

The month flashes on the display.

- Select the desired month using the memory button **M1** or **M2** and confirm with the **START/STOP** button **⓪**.

The day flashes on the display.

- Select the desired day using the memory button **M1** or **M2** and confirm with the **START/STOP** button **⓪**.

- i** If the hour format is set as **12h**, the day/month display sequence is reversed.

The hours flash on the display.

Select the desired hour using the memory button **M1** or **M2** and confirm with the **START/STOP** button **ⓘ**.

The minutes flash on the display.

• Select the desired minute using the memory button **M1** or **M2** and confirm with the **START/STOP** button **ⓘ**.

The *Bluetooth*® symbol is shown in the display.


• Use the memory button **M1** or **M2** to select whether automatic *Bluetooth*® data transfer is activated (*Bluetooth*® symbol is displayed) or deactivated (*Bluetooth*® symbol is not shown) and confirm with the **START/STOP** button **ⓘ**.

i *Bluetooth*® transfers will reduce the battery life.

7. Usage

Ensure the device is at room temperature before measuring. The measurement can be performed on the left or right wrist.

Attaching the cuff

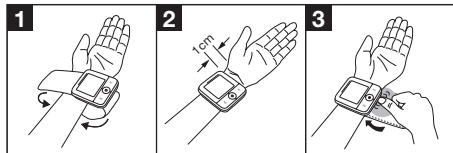- Expose your wrist. Ensure that the circulation of the arm is not hindered by tight clothing or similar.
- Place the cuff on the inside of your wrist.
- Close the cuff with the hook-and-loop fastener such that the upper edge of the device is positioned approx. 1 cm underneath the heel of the hand.
- The cuff must be fitted tightly around the wrist, but must not constrict.

Blood pressure may vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm.

Adopt the correct posture

- Before the initial blood pressure measurement, make sure always to rest for about 5 minutes. Otherwise deviations can occur.
- You can take the measurement while sitting or lying. To take your blood pressure, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground. Make sure to rest your arm and move it. Always make sure that the cuff is at heart level. Otherwise significant deviations can occur. Relax your arm and the palm of your hand.

- **i** To make the measurement more comfortable, you can place your lower arm on the storage box while taking the measurement. To do so, place your lower arm on the notch on the storage box as shown.
- To avoid falsifying the measurement, it is important to remain still during the measurement and not to speak.

Selecting memory

You have 2 memories, each with 60 memory spaces, to enable the separate storage of measurements for 2 different people.

To select the corresponding user memory, press the **M1** memory button (for user ) or **M2** (for user ) on the switched-off device. Confirm your selection by pressing the **START/STOP** button .

Performing the blood pressure measurement


As described above, attach the cuff and adopt the posture in which you want to perform the measurement.

- Press the **START/STOP** button  to start the blood pressure monitor. All displays will illuminate briefly.


The blood pressure monitor will begin the measurement automatically after 5 seconds.

The cuff automatically inflates. The measurement itself is taken during the inflation phase. As soon as a pulse is found, the pulse symbol  is displayed.

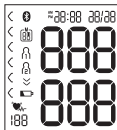- **i** You can cancel the measurement at any time by pressing the **START/STOP** button .

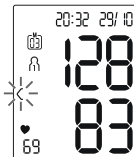- The systolic pressure, diastolic pressure and pulse rate measurements are displayed.

- **Er_** appears if the measurement could not be performed properly. Observe the chapter „9. What if there are problems?“ in these instructions for use and repeat the measurement.
- Press the **START/STOP** button  to switch off the blood pressure monitor. The measurement is then stored in the selected user memory. If **Bluetooth® data transfer** is activated, data is transferred after pressing the **START/STOP** button . The **Bluetooth®** symbol flashes on the display. The blood pressure monitor now attempts to connect to the app for approx. 30 seconds. The **Bluetooth®** symbol stops flashing as soon as a connection is established. All measurement data is automatically transferred to the app. Once the data has been successfully transferred, the device switches off.
- If a connection to the app cannot be established after 30 seconds, the **Bluetooth®** symbol goes out and the

Er_

blood pressure monitor switches off automatically after 5 seconds.

- If you forget to turn off the device, it will switch off automatically. In this case too, the value is stored in the selected or most recently used user memory.
- Wait for at least 1 minute before taking another measurement.

Evaluating results

Cardiac arrhythmia:

This device can identify potential disruptions of the heart rhythm when measuring and if necessary, indicates this after the measurement with the symbol .

This can be an indicator for arrhythmia. Arrhythmia is an illness in which the heart rhythm is abnormal because of flaws in the bioelectrical system that regulates the heart-beat. The symptoms (skipped or premature heart beats, pulse being slow or too fast) can be caused by factors such as heart disease, age, physical make-up, excess stimulants, stress or lack of sleep. Arrhythmia can only be determined through an examination by your doctor.

If the symbol  is shown on the display after the measurement has been taken, repeat the measurement. Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol  appears frequently, please consult your doctor. Self-

diagnosis and treatment based on the measurements can be dangerous. Always follow your GP's instructions.

Risk indicator:

The measurements can be classified and evaluated in accordance with the following table.

However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The bar chart on the display and the scale on the unit show which category the recorded blood pressure values fall into. If the values of systole and diastole fall into two different categories (e.g. systole in the 'High normal' category and diastole in the 'Normal' category), the graphical classification on the device always shows the higher category; for the example given this would be 'High normal'.

Blood pressure value category		Systole (in mmHg)	Diastole (in mmHg)	Action
Setting 3: severe hypertension	red	≥ 180	≥ 110	seek medical attention
Setting 2: moderate hypertension	orange	160–179	100–109	seek medical attention
Setting 1: mild hypertension	yellow	140–159	90–99	regular monitoring by doctor
High normal	green	130–139	85–89	regular monitoring by doctor
Normal	green	120–129	80–84	self-monitoring
Optimal	green	< 120	< 80	self-monitoring

Source: WHO, 1999 (World Health Organization)

Saving, displaying and deleting measured values

User memory

The results of every successful measurement are stored together with the date and time. The oldest measurement is overwritten in the event of more than 60 measurements.

- To select the corresponding user memory, press the **M1** memory button (for user **A**) or **M2** (for user **B**) on the switched-off device. Confirm your selection by pressing the **START/STOP** button **ⓘ**.
 - If *Bluetooth*[®] is activated (the **ⓑ** symbol flashes on the display), the blood pressure monitor attempts to connect to the app. The buttons are deactivated and the **ⓑ** symbol ceases to flash as soon as a connection is established and the data is transferred.
- ⓘ** If you press the memory button **M1** or **M2** in the meantime, the transfer is cancelled. The **ⓑ** symbol is no longer shown.

Average values

A is displayed.

The average value of all saved measured values in this user memory is displayed.

- Press the memory button **M1**.

AM is displayed.

The average value of the morning measurements for the last 7 days is displayed (morning: 5.00 a.m. – 9.00 a.m.).


- Press the memory button **M1**.

PM is displayed.

The average value of the evening measurements for the last 7 days is displayed (evening: 6.00 p.m. – 8.00 p.m.).


- If you press the memory button **M1** again, the last individual measurement is displayed (in this example, measurement 03).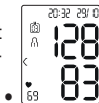

- If you press the memory button **M1** once more, you can view your individual measured values.
 - To switch the device off again, press the **START/STOP** button ①.
- ① You can exit the menu at any time by pressing the **START/STOP** button ①.

- To erase a user's memory, first select the user memory to be erased by pressing the **M1** or **M2** memory button on the off device and confirm your selection by pressing the **START/STOP** button ①.
- The average value of all measurements appears on the display; in parallel, **A** lights up.
- Press and hold down both memory buttons **M1** and **M2** simultaneously for 5 seconds.

CL 00 appears on the display. All the values in the selected user memory have now been deleted.

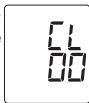Transfer via **Bluetooth®**

It is also possible to transfer the measured values saved on the device to your smartphone via **Bluetooth®**. You will need the “beurer HealthManager” app for this. This is available in the Apple App Store and from Google Play.

System requirements for the app “beurer HealthManager”:

- iOS ≥ 10.0, Android™ ≥ 5.0
- **Bluetooth®** ≥ 4.0

List of compatible devices:

Proceed as follows to transfer values:

If **Bluetooth®** is activated in the settings menu, the data is transferred automatically after the measurement.

The **B** symbol appears in the top left of the display (see chapter „6. Initial use“, section “**Bluetooth®**”).

- ① When connecting for the first time, a randomly-generated six-digit PIN code is displayed on device, and at the same time an input field appears on the smart-

phone in which you must enter this six-digit PIN code. After successfully entering the code, the device will be connected to your smartphone.

Step 1: BC 54

Activate *Bluetooth*® on your device (see chapter „6. Initial use“, section “*Bluetooth*®”).

Step 2: “beurer HealthManager” app

In the “beurer HealthManager” app, add the BC 54 under “Settings/My devices”.

Step 3: BC 54

Take a measurement.

Step 4: BC 54:

Data transfer immediately following measurements.

- If **Bluetooth**® **data transfer** has been activated, data is transferred after having confirmed the user memory and pressing the **START/STOP** button **ⓘ**.

Step 4: BC 54:

Data transfer at a later point:

- Go to memory mode (chapter „7. Use“, section „Saving, displaying and deleting measured values“). Select the desired user memory. The *Bluetooth*® transfer starts automatically.

- ⓘ** The “beurer HealthManager” app must be active to allow data transfer.

If your smartphone has a protective cover, remove this to ensure that there is no interference during the transfer. Begin the data transfer in the “beurer HealthManager” app.

We would explicitly draw attention to the fact that the software to hand is not a medical product in accordance with EU Directive 93/42/EEC.

The values displayed are purely for visualisation purposes and must not be used as a basis for therapeutic treatments. The software is not part of a diagnostic-medical system.

8. Cleaning and maintenance

- Clean the device and cuff carefully using a slightly damp cloth only.
- Do not use any cleaning agents or solvents.
- Under no circumstances hold the device and cuff under water, as this can cause liquid to enter and damage the device and cuff.
- If you store the device and cuff, do not place heavy objects on the device and cuff. Remove the batteries.

9. What if there are problems?

Error message	Possible cause	Solution
Er1	Unable to record a pulse.	Please wait one minute and repeat the measurement.
Er2	You have moved or spoken during the measurement.	Ensure that you do not speak or move during the measurement.

Error message	Possible cause	Solution
Er3	The cuff was not attached correctly.	Please observe the notes in chapter "7. Usage" under the section "Attaching the cuff". During a new measurement, check whether the cuff can now be inflated properly.
Er4	An error occurred during measurement.	Please wait one minute and repeat the measurement.
Er5	The inflation pressure is higher than 300 mmHg or the measured blood pressure is outside the measuring range.	Ensure that you do not speak or move during the measurement.
Er6	The batteries are almost empty.	Insert new batteries into the device.

Error message	Possible cause	Solution
E-7	The data could not be sent via <i>Bluetooth®</i> .	Proceed as described in chapter „7. Usage“ under the section „Transfer via <i>Bluetooth®</i> “ into the manual memory mode and try to restart the data transfer.
E-8	A device error has occurred.	Please repeat the measurement after a pause of one minute.

If the problems persist despite the proposed corrective actions, please contact customer service.

10. Disposal

Repairing and disposing of the device

- Do not repair or adjust the device yourself. Proper operation can no longer be guaranteed in this case.
- Do not open the device. Failure to comply will result in voiding of the warranty.
- Repairs must only be carried out by Customer Services or authorised suppliers. Before making a claim, please check the batteries first and replace them if necessary.

- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

Disposing of the batteries

- Batteries must not be disposed of with household waste. They may contain toxic heavy metals and are subject to special waste treatment.
- The codes below are printed on batteries containing harmful substances:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.

11. Technical specifications

Model no.	BC 54
Measurement method	Oscillometric, non-invasive blood pressure measurement on the wrist
Measurement range	Cuff pressure 0-299 mmHg, Systolic 60-230 mmHg, Diastolic 40-130 mmHg, Pulse 40-199 beats/minute

Display accuracy	Systolic ± 3 mmHg, Diastolic ± 3 mmHg, Pulse ± 5 % of the value shown
Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: Systolic 8 mmHg / Diastolic 8 mmHg
Memory	2 x 60 memory spaces
Dimensions	L 80.5 mm x W 69.5 mm x H 25 mm
Weight	Approximately 116 g (without batteries, with cuff)
Cuff size	135 to 215 mm
Permissible operating conditions	+5 °C to +40 °C, 15-90 % relative humidity (non-condensing), 700-1060 hPa ambient pressure
Permissible storage and transport conditions	-20 °C to +60 °C, $\leq 93\%$ relative humidity (non-condensing)
Power supply	2 x 1.5V --- AAA batteries
Battery life	For approx. 150 measurements, depending on levels of blood pressure and pump pressure
Software version	A01

Data transfer	2402 MHz – 2480 MHz frequency band Transmission power max. 7 dBm The blood pressure monitor uses <i>Bluetooth</i> [®] low energy technology Compatible with <i>Bluetooth</i> [®] ≥ 4.0 smartphones/tablets
---------------	--

The serial number is located on the device or in the battery compartment.

Technical information is subject to change without notification to allow for updates.

- This device is in line with European Standard EN 60601-1-2 (In accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3 and IEC 61000-4-8) and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit.
- This device corresponds to the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktgesetz) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment –

Part 2-30: Particular requirements for the basic safety and essential performance of automated non-invasive sphygmomanometers).

- The accuracy of this blood pressure monitor has been carefully checked. Calibration is not necessary.
- The device has been developed with regard to a long useful life. The expected service life is 5 years.
- If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.
- We hereby confirm that this product complies with the European RED Directive 2014/53/EU. The CE Declaration of Conformity for this product can be found under: www.beurer.com/web/we-landingpages/de/cedeclearationofconformity.php

12. Warranty / Service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 5 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist aris-

ing from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments and so on.

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment (BC 54), including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

If any: a list of all cables and maximum lengths of cables (if applicable), transducers and other ACCESSORIES that are replaceable by the RESPONSIBLE ORGANIZATION and that are likely to affect compliance of the ME EQUIPMENT or ME SYSTEM with the requirements of Clause 7 (EMISSIONS) and Clause 8 (IMMUNITY). ACCESSORIES may be specified either generically (e.g. shielded cable, load impedance) or specifically (e.g. by MANUFACTURER and EQUIPMENT OR TYPE REFERENCE). If any: the performance of the ME EQUIPMENT or ME SYSTEM that was determined to be ESSENTIAL PERFORMANCE and a description of what the OPERATOR can expect if the ESSENTIAL PERFORMANCE is lost or degraded due to EM DISTURBANCES (the defined term "ESSENTIAL PERFORMANCE" need not be used).

Table 1

Guidance and manufacturer's declaration – electromagnetic emissions	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not application
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not application

Table 2

Guidance and manufacturer's declaration – electromagnetic immunity		
Immunity test	IEC 60601-2 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	power supply lines: ±2 kV input/output lines: ±1 kV	Not application
Surge IEC61000-4-5	line(s) to line(s): ±1 kV line(s) to earth: ±2 kV 100 kHz repetition frequency	Not application

Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% U_T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U_T ; 1 cycle and 70% U_T ; 25/30 cycles Single phase: at 0° 0% U_T ; 300 cycle	Not application
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	150KHz to 80MHz: 3Vrms 6Vrms (in ISM and amateur radio bands) 80% Am at 1kHz	Not application

Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz
NOTE U_T is the a.c. mains voltage prior to application of the test level.		

Table 3

Guidance and manufacturer's declaration - electromagnetic immunity							
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
	385	380-390	TETRA 400	Pulse modulation b) 18Hz	1.8	0.3	27
	450	430-470	GMRS 460: FRS 460	FM c) \pm 5kHz deviation 1kHz sine	2	0.3	28
	710	704-787	LTE Band 13, 17	Pulse modulation b) 217Hz	0.2	0.3	9
	745						
	780						
	810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation b) 18Hz	2	0.3	28
	870						
	930						

	1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulati- on b) 217Hz	2	0.3	28
	1845						
	1970						
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulati- on b) 217Hz	2	0.3	28
	5240	5100-5800	WLAN 802.11 a/n	Pulse modulati- on b) 217Hz	0.2	0.3	9
	5500						
	5785						

The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer GmbH is under license. Other trademarks and trade names are those of their respective owners. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.

Guangdong Transtek Medical Electronics Co., Ltd. Zone A, No.105, Dongli Road,
Torch Development District, Zhongshan, 528437, Guangdong, China

MDSS - Medical Device Safety Service GmbH, Schiffgraben 41, 30175 Hannover, Germany

Distributed by: BEURER GmbH, Söflinger Str. 218, 89077 Ulm (Germany)
www.beurer.com, www.beurer-blutdruck.de
www.beurer-gesundheitsratgeber.com, www.beurer-healthguide.com