beurer _{medical}	FT 09	 0483	 Beurer GmbH · Söflinger Straße 218 · D-89077 Ulm · GERMANY Tel.: 0731/3989-144 · Fax: -255 · E-Mail: kd@beurer.de	beurer _{medical}	FT 09	 0483	 Beurer GmbH · Söflinger Straße 218 · D-89077 Ulm · GERMANY Tel.: 0731/3989-144 · Fax: -255 · e-mail: kd@beurer.de
DEUTSCH							

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Important safety information – retain for later use.

- In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.
- The thermometer is intended to measure the temperature of the human body.
- The FT 09 thermometer is only designed for the measuring area on the human body stated in the instructions for use.
- Children should not use the thermometer without supervision.
- Before each use, check the thermometer for signs of damage or wear. Damaged or worn thermometers should not be used.
- Always wait for the acoustic signal which indicates the minimum time required for measurement. Discuss the temperature with your family physician.
- The thermometer contains sensitive electronic parts. Therefore, protect it from blows, bending, high temperature or direct exposure to sunlight.
- Using the thermometer in strong electromagnetic fields such as next to a mobile phone, can cause a malfunction.
- When the thermometer is switched on, it carries out a self-test. Therefore the accuracy of measurement need not be tested at regular intervals.
- If the temperature is very abnormal, please contact your family doctor promptly.
- This device meets the requirements of the EU Directive 93/42/EC concerning medical devices, as well as those of the Medizinproduktegesetz (German Medical Devices Act), the European Standard EN 12470-3: Clinical thermometers – part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device and the European Standard EN60601-1-2, and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.

Measuring methods: Temperature measurement in the anus (rectal)

This method of measurement is most reliable and most accurate. Guide the tip of the thermometer carefully 2 to 3 cm into the anus.

beurer _{medical}	FT 09	 0483	 Beurer GmbH · Söflinger Straße 218 · D-89077 Ulm · GERMANY Tel.: 0731/3989-144 · Fax: -255 · e-mail: kd@beurer.de	beurer _{medical}	FT 09	 0483	 Beurer GmbH · Söflinger Straße 218 · D-89077 Ulm · GERMANY Tel.: 0731/3989-144 · Fax: -255 · e-mail: kd@beurer.de
FRANÇAIS							

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Electromagnetic Compatibility Information

<i>Guidance and manufacturer’s declaration – electromagnetic emissions</i>		
The FT 09 is intended for use in the electromagnetic environment specified below. The customer or the user of the FT 09 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The FT 09 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The FT 09 is suitable for use in all establishments, including domestic establishments and those directly con-nected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

beurer _{medical}	FT 09	 0483	 Beurer GmbH · Söflinger Straße 218 · D-89077 Ulm · GERMANY Tel.: 0731/3989-144 · Fax: -255 · E-Mail: kd@beurer.de	beurer _{medical}	FT 09	 0483	 Beurer GmbH · Söflinger Straße 218 · D-89077 Ulm · GERMANY Tel.: 0731/3989-144 · Fax: -255 · e-mail: kd@beurer.de
ITALIANO							

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Measurement in the mouth cavity (oral)

For this purpose, guide the tip of the thermometer carefully into one of the heat pouches beneath the tongue, to the left or the right of the root of the tongue.

Measurement of temperature in the armpit (axillary)

This method of measurement is relatively inaccurate. Therefore, it cannot be recommended from a medical point of view.

Mode of measurement	Recommended time required for measurement	Mean divergence to the rectal method
in the anus	until the acoustic signal	–
in the oral cavity	until the acoustic signal	-0,4 °C to -1,5 °C
in the armpit	minimum 10 minutes	-0,7 °C to -2,0 °C

Use

First remove the protective foil from the display. For switching on, briefly press the on/off key. A short acoustic signal will announce that the thermometer is switched on. The thermometer will carry out a self-test for about 2 seconds. During the selftest all segments of the display will appear. Then the reference reading (36,5 °C) appears. When you release the button, the flashing symbol °C° appears and the symbol «L» is shown in the display screen. The thermometer is now ready for use. During the measurement, the current temperature is displayed continuously, and the °C° symbol flashes on and off. The measurement is completed when a stable temperature has been reached. When this stage has been reached, an acoustic signal is heard, the °C° symbol stops flashing and the temperature measured is displayed. As long as the measured temperature is below 32 °C, «L»°C° will appear. Higher temperatures will be shown directly, e.g. 32.1 °C. By prolonging the length of measurement until after the acoustic signal, more accurate measurements can be obtained. In case the temperature is measured in the armpit, the thermometer should be left in the armpit for the recommended time of measurement, in spite of the acoustic signal. If temperatures over 37,5 °C are measured, a fever alarm sounds (short bleeping tones, 4 seconds long). If temperatures higher than 44 °C are measured, the symbol for «H °C» will appear. To prolong the life of the battery,

beurer _{medical}	FT 09	 0483	 Beurer GmbH · Söflinger Straße 218 · D-89077 Ulm · GERMANY Tel.: 0731/3989-144 · Fax: -255 · e-mail: kd@beurer.de	beurer _{medical}	FT 09	 0483	 Beurer GmbH · Söflinger Straße 218 · D-89077 Ulm · GERMANY Tel.: 0731/3989-144 · Fax: -255 · E-Mail: kd@beurer.de
ITALIANO							

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